

3
New
Features

cura¹
Helping To Care

DoorAlert™



Wandering Resident Detection System

DoorAlert™ is the complete Wandering Resident Detection System that helps you provide better care for your wandering residents with less burden on carers and staff.

cura¹, the makers of DoorAlert™ understand the pressures placed on caregivers responsible for the safety and quality of life of residents who are prone to wander.

How do you improve the safety of wandering residents without overly restricting their movement, reducing the attractiveness of an open environment for visitors and non-wanderers, whilst at the same time, lowering the need for carer intervention?

DoorAlert™ makes this all possible by unobtrusively monitoring and managing egress to almost any exit such as doors, elevators, gates, manual and automatic sliding doors etc.

The flexible configuration makes DoorAlert™ suitable for any kind of institution and even home use.

So how exactly does DoorAlert™ work?

Consider these 4 scenarios as a wandering patient approaches a monitored exit. The DoorAlert™ system senses the approaching wandering resident and ...

Scenario 1

If the wandering resident comes within 1 metre (adjustable) of the door monitor, the system silently locks the door. The door is automatically unlocked when the resident moves out of the detection zone.

Staff access is possible during locked period via SmartGuard Keypad, Caregiver Key, Swipe Card etc.

Scenario 2

If the wandering resident opens the door, the system activates its alarm mode to alert caregivers.

Scenario 3

If the wandering resident has moved through a monitored exit or corridor then an alarm condition is triggered on a Desktop Care Station, identifying the wandering resident and where they are trying to exit.

Scenario 4

An alarm condition can be triggered via a connected nurse call system, alerting staff and caregivers.



3 New Features

Night Mode – To protect a facility at night, any monitored door opened will alert staff/ caregivers.

Dry Contact Relay Outputs (optional) – For easy connection to existing nurse call systems and other third party alarm equipment.

Latching Lock – Doors remain locked while the resident is within the vicinity of the DoorAlert™ Monitor (no time out) unless overridden by staff.

Standard Features

- Easy installation, configuration and operation.
- Easily expandable and upgradable.
- Monitor from 1 to 1,000 residents.
- Low maintenance.
- Adjustable detection range to suit facility conditions.
- Individually identifies each resident.
- Fully wireless – no cabling required.
- Flashing LED and adjustable audible alarm.
- Can be connected to a nurse call system and electronic locking devices.
- Caregiver Reset Key, Smartguard Keypad and Reset Transmitter options.
- Low battery indicator.
- Magnetic door contact input can inhibit alarm activation if the door is closed.



Nursecall System



3501 ActiveCare Pager

2552 Door Monitor



3500 ActiveCare Console



2501 Desktop Care Station

- SMS
- Pagers
- Annunciators
- Email





cura¹
Helping To Care

To find out more about DoorAlert™
including Technical Specifications, visit
our website at **www.cura1.com**

Or call us today on **1300 12 22 73**
(Distribution and installation specialists in every state.)



Our products are supplied with a full warranty and are
guaranteed to be free from defects in material and workmanship.

cura1 products are designed to augment well designed care programs & practices, cura1 does not claim that any of its products are suitable as a substitute, or as a total solution for individuals that require care and constant supervision & are at risk.